


PROPOLIS therapy

CLEANING & MAINTENANCE

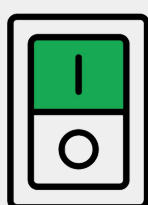

SAFETY FIRST

Switch off the device before each cleaning operation and disconnect the mains cable from the socket.

DIFFUSER CLEANING

Use a damp cloth with antibacterial cleanser for the device and exterior cleaning of connecting tube



ACCESSORIES SANITIZING (MASK AND TUBE)



Possibility of various solutions:

- 1) In hot water (about 40° C) with mild dish detergent;
- 2) By immersion in a solution of water and vinegar, in the proportions of $\frac{3}{4}$ and $\frac{1}{4}$, respectively, or in water and sodium bicarbonate;
- 3) Soak in cold water with a small glass of disinfectant for about 15 minutes.

RINSING AND FINAL DRYING

After cleaning, the components must be drained well on a clean surface. Then dry with a soft cloth. Drying is important because germs grow in humidity; for this reason, before storing the aerosol, make sure it is dry.



USE OF THE MASK

The mask is for personal use, therefore it can be used by a single patient, to get the maximum benefit and avoid the spread of germs and bacteria.